# mobee

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Thank you for purchasing 

### 1. First, you should know:

This start-up booklet will familiarize you with your new product step by step. However, it does not represent a complete product documentation. The software you purchased includes an entirely digitalized and comprehensive user guide. In this way, we protect the environment and ensure that you always have the newest user guide in case of any updates.

The booklet is limited to the information and steps necessary before starting to work with the system.

In order to view the complete user guide you need a PDF Reader. Please make sure such a Reader is installed on your computer. In case you should not have a Reader, it can be downloaded for free from various providers on the internet.

### 2. Contents of your mobee set ... is it complete?

In addition to the booklet, you should have received the following components:

- 1 **mobee** measuring device (**med:** blue, **fit:** orange)
- **mobee** PC software on USB Card
- Fastening system consisting of 3 elastic fastening belts, each with one belt clip
- 1 radio stick for wireless communication (**mobee com**)
- 1 charging cable USB to micro-USB
- 1 power plug 230V with USB connection (country-specific, this part may not be included)
- 1 USB extension cable 2.0 A-A, 180cm
- 1 Transport case (**med:** white, **fit:** black)

### Is your set complete?

If so, continue with the next step.

In case something is missing, please consult your distributor.

### 3. System Requirements – what your PC needs:

You will need a Windows®-compatible PC to use the **mobee** software. Moreover, your system should meet the following minimum requirements:

- CPU (=computer performance) 1.5 GHz
- RAM (=random access memory) 2048 MB
- Min. display resolution 1280x720
- 3 GB of available hard disk space
- 1 free USB slot
- PDF Reader

Windows 7 and later operating systems are recommended when using the **mobee** software.

### 4. Important safety instructions – keep away from:

- Damaged cables or plugs
- Measuring systems with damaged body
- Charging devices that do not belong to the set
- Own attempts at battery change

In these cases, please contact your distributor!

### 5. For a properly functioning and durable system:

- Preferably use and store the system at temperatures between 0°C and +40°C and a relative humidity of between 45 and 85%.
- Avoid dropping or hitting the device.
- Keep the micro-USB port sealed while the measuring system is not charging.
- Apart from wipe disinfection, do not perform any other type of disinfection.

## 6. Start-up preparation ... ready, set, go!

*Support from distribution partners:*

Depending on country and contract, you may be eligible for support services for your system start-up and for an introduction into the operation. Ask your distributor about possibilities.

Fully charge the battery of the device with the supplied USB charging system before you start up the system. To plug in the charging cable, carefully open the rubber plug at the bottom edge of the measuring device. The battery is fully charged when both LEDs of the measuring device are continuously illuminated.

Select the computer you would like to run the software on. Make sure that you have administrator rights on the selected computer and that the system requirements are fulfilled.

If you have bought both, **mobee med** and **mobee fit**, it makes no difference which product you install first.

Now, perform the following steps:

- Plug the provided USB card into an available USB port of your computer, open Windows Explorer and navigate to the USB drive. Run the file *CDStart.exe*.
- Select the option »User Manual«.
- Afterwards, proceed with the rest of the installation and the start-up as described in the user guide.

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**SportMed** AG