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## Start-up booklet

Thank you for purchasing a product of the **mobee**<sup>360</sup> platform!

### 1. First, you should know:

This start-up booklet will familiarize you with your new product step by step.

However, it does not represent a complete product documentation. It is provided together with the software in a digital user manual.

This booklet is limited to the information and steps necessary **before** starting to work with the system.

In this way, we protect the environment and ensure that you always have the latest user guide at hand in case of any updates and changes.

## 2. System requirements - what your PC needs:

You will need a Windows®-compatible PC to use the software. Moreover, your system should meet the following minimum requirements:

- CPU (= computer performance) 2 GHz
- RAM (= random access memory) 4096 MB
- Min. display resolution 1280x720
- 3 GB of available hard disk space
- min. 1 free USB slot
- PDF Reader for displaying printouts and digital user documentation

A Windows 7 operating system and newer is required to use the software.

### **3. Important safety instructions – keep away from:**

- Damaged cables or plugs
- Devices with damaged body
- Charging devices not belonging to the set
- Own attempts at battery change  
(on devices with permanently installed battery)

In these cases, please contact your distributor!

### **4. For a properly functioning and durable system:**

- Avoid dropping or hitting the device.
- Apart from wipe disinfection, do not perform any other type of disinfection.
- Observe the transport, storage and use instructions in the respective product documentation.

### **5. Installation:**

- The software is located on the mobee®360 USB drive.
- Make sure that you have sufficient administrator rights for the PC on which you want to install the software!

## 6. Activation

- The license key required for activation can be found on the mobee<sup>®</sup>360 USB drive.
- Store the document in a safe place! Among other things, it contains your customer number (Lissy ID), which you need to use the support hotline.

## 7. Help

On our YouTube channel "mobee<sup>®</sup>360 - Digitale Assessments für Gesundheitsmacher" you will find various instructions, e.g. how to activate the software.

You can also find instructions and support documents in your mobee<sup>®</sup> software under mobee<sup>®</sup>360 > Help.

### 7.1 Support from distribution partners

Depending on country and contract, you may be eligible for support services for your system start-up and for an introduction into the operation. Ask your distributor about possibilities.

### Content of the USB drive

- Software
- License key (pdf document)
- Help documents (FAQ etc.)  
"Data" folder > Documents
- Basic marketing kit

## 8. Special information for users of mobee®med & mobee®fit systems

### Contents of your mobee® set... is it complete?

In addition to the booklet, you should have received the following components:

- 1 mobee measuring device (**med:** blue, **fit:** orange)
- mobee PC software on USB device
- 3 elastic fastening belts, each with one belt clip
- 1 USB radio stick for wireless communication (mobee com)
- 1 charging cable USB to micro-USB
- 1 power plug 230 V with USB connection (country-specific, this part may not be included)
- 1 USB extension cable 2.0 A-A, 180 cm
- 1 transport case

## 9. Is your set complete?

If so, continue with the next step.

In case something is missing, please consult your distributor.

### **Please note:**

The USB extension cable is located in the case lid behind the foam cover!

Fully charge the battery of the mobee® device with the supplied USB charging system before you start up the system.

To plug in the charging cable, carefully open the rubber plug at the bottom edge of the measuring device.

The battery is fully charged when both LEDs of the measuring device are continuously illuminated.

# mobee 360

SportMed AG  
Soci t  anonyme

46, route de Wasserbillig • L-6490 Echternach  
www.mobee.de • info@mobee.de